Italiano

TERMOMETRO DIGITALE - TYPE DMT4218

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STANDARDS

Il prodotto risponde ai seguenti standards: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance — Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

	Prodotto da: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China
	Shanghai International Corp. GmbH (Europe) Effestrasse 80, 20537 Hamburg, Germany
	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug - Switzerland

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EN English

DIGITAL THERMOMETER TYPE DMT4218

Before use, read the instructions and warnings given in this handbook carefully and keep the handbook in a safe place. Digital thermometer with a large LCD display for easy reading. It measures body temperature in 60 seconds. Fast and accurate fitted with a sound signal when the temperature reading has been taken. Stores the last read temperature, to evaluate any variations. Extremely easy to use, it is kept in a practical case to avoid risk of breaking. With washable flexible probe and automatic switching-off function.

KET TO SYMBOLS		
 Caution! Read the user instructions carefully	 Symbol for "Type BF applied parts"	
	In compliance with European legislation on medical devices	
	Manufacturer	
	European Representation	
	Swiss representative	
	Batch production number	

IP27: Degree of protection of electrical equipment covers, where the first figure indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second figure the degree of protection against penetration by liquids (from 0 to 8).

GENERAL WARNINGS

Self-measuring means control, not diagnosis and treatment, any unusual readings must be discussed with your doctor. Under no circumstances must you change the medicine doses that have been prescribed by your doctor. 2) Do not squeeze, bend, drop, bite or break the thermometer. 3) Keep it out of the reach of children. 4) Do not expose to sunrays or direct sources of heat. Keep in a dry clean place. 5) Do not immerse the thermometer in water or other liquids, only the probe is waterproof. 6) Do not take the measurement when the thermometer is wet because the results could be inaccurate. 7) This thermometer reads only the armpit, rectal or oral temperature. Do not use it for reading auricular temperatures. 8) Do not drink hot or cold drinks, do not smoke, do not take a shower before measuring body temperature. 9) Do not use the thermometer orally after it was used rectally without disinfecting it first. 10) For children under two years of age, do not use the thermometer orally. 11) The device may give inaccurate readings if it is used in temperatures or humidity levels outside the limits indicated in the "Technical Specifications" paragraph. 12) Do not use near strong magnetic fields and therefore keep away from radio or mobile phone systems (for more information on interferences, see the paragraph "Electromagnetic compatibility information").

TECHNICAL SPECIFICATIONS

- Product name: digital thermometer
- Commercial name: TH3601
- Operating mode: Direct Mode
- Measuring range: 32.0°C - 42.9°C
- Accuracy: +/-0,1°C (35,5°C - 42,0°C) ambient operating range between 18°C and 28°C; +/- 0,2°C (<35,5°C, >42,0°C) ambient operating range <18°C and >28°C
- Display: liquid crystal
- Battery: 1x1,5V type LR/SR-41
- Battery autonomy: approx. 200 hours continuous use
- Environmental conditions for use: from +5°C to +40°C; relative humidity between 15% and 95% RH
- Environmental conditions for storage: from -20°C to +55°C; relative humidity between 15% and 95% RH

INSERTING/REPLACING THE BATTERIES

1) Pull the battery cover in the direction shown on the arrow. 2) Carefully extract the battery compartment to the right to ease the battery removal. Use a pointed object to remove the old batteries. 3) Place the new batteries with the positive pole (+) pointed upwards and the negative pole pointed downwards. 4) Gently push the battery compartment into its seat and re-assemble the cover. Should the display show the symbol , it means that they have to be replaced. 5) Remove the use batteries as described under points 1-2 and dispose of them as special waste at the collection points for recycling.

For further information concerning the disposal of exhausted batteries, refer to the store where the device containing the batteries was purchased, the Municipality or the local waste disposal service.

Caution: For safety reasons, it is recommended never to allow children younger than 12 years to remove the batteries.

INSTRUCTIONS FOR USE

Before and/or after each use, disinfect the probe using alcohol.

"ORAL AND RECTAL USE

1) Switch on the thermometer by pressing the O/I key, the device emits a short acoustic signal. On the display, all function symbols switch on, just for few seconds. Then, the thermometer shows for few seconds the last measurement performed, followed by 32°C or Hi°C (if the detected temperature is lower than 32°C) or Hi°C (if the detected temperature is greater than 42,9°C). When the symbol °C blinks, the device is ready for use.

2) Position the probe as follows:

ORAL USE: before the measurement close the mouth for about 5 minutes to stabilize the temperature. Position the probe under the tongue and close the mouth till the thermometer measures the temperature. The standard temperature for this type of measurement is 37°C.

RECTAL USE: lubricate the probe using a jelly water-soluble solution. Do not use Vaseline. Insert the probe in the rectum for about 1 cm till temperature stabilizes (about 60 seconds). The standard temperature for this type of measurement is 37,5°C. Do not force the thermometer in the rectum. If insertion causes pain, immediately stop the measurement. Failure to follow this precaution could cause injury.

"UNDERARM USE

For underarm use, proceed as follows:

1) Dry the underarm very well. Before the measurement keep the arm along the hip for 5 minutes to stabilize the temperature. Position the probe under the underarm and keep the arm well pressed against the chest. (This prevents the room temperature, in which one is, from influencing the correct measurement of body temperature).

2) Press the key O/I. The device emits a short acoustic signal. On the display all function symbols do switch on, just for few seconds. Then, the thermometer shows for few seconds the last measurement performed, followed by Lo°C (if the detected temperature is lower than 32°C) or Hi°C (if the detected temperature is greater than 42,9°C). When the symbol °C blinks, the device is ready for use. The temperature stabilizes after about 60 seconds. The standard temperature for this type of measurement is 36,5°C.

3) When the device emits the acoustic signal, remove it. The measurement has ended and it is shown on the display. When the detected temperature exceeds 37,8°C, the intensity and frequency of the acoustic signal increase.

The device switches off automatically after 10 minutes, if it is not used (meanwhile the measurement remains displayed); otherwise, it can be switched off using the key O/I.

MAINTENANCE

Clean the thermometer probe using a cloth moistened with alcohol or with any type of topic disinfectant before and/or after each use. Any other type of solution can damage the device. The thermometer body has to be cleaned using a dry cloth. Do not immerse the thermometer in water or other liquids. Never use chemical or abrasive products. The probe is the most fragile part of the thermometer. Pay utmost care during its cleaning. Do not use alcohol to clean the display. Preserve the thermometer in a dry and clean place, away form direct sunlight. Do not sterilize or boil the thermometer. Preserve the thermometer carefully in its standard-supplied casing. It is recommended the performance should be checked every 2 years. Please contact the service Laica (this is not included in warranty).

DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

 The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment **(Dir. 2012/19/Eu-WEEE)**. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions.To correctly dispose of batteries **(Dir. 2013/56/UE) do not throw them into household waste** but rather dispose of them as special waste at specialised recycling centres. For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

WARRANTY

This appliance is guaranteed for 2 years from the time of delivery of the goods, or another longer term envisaged by the national legislation of the consumer's residence. This provision complies with Italian and European legislation. The Laica products are designed for home use and must not be used in public venues. The warranty only covers manufacturing defects and does not apply if the damage is caused by an accidental event, incorrect use, negligence or misuse of the product. Use only the accessories supplied; the use of different accessories may result in invalidity of the warranty. Do not open the unit for any reason; in the case of opening or tampering, the warranty is definitively voided. This warranty does not apply to parts subject to wear or to the batteries when supplied. After 2 years from delivery, or another longer term envisaged by the national legislation of the consumer's residence, the warranty expires; in this case, the technical assistance interventions will be carried out against a fee. Information on technical assistance, whether under warranty or for a fee, can be requested by contacting info@laica.com.

No payment will be due for repairs or replacements of products that fall within the terms of the warranty. In the event of faults, contact the retailer. Do NOT send the appliance directly to LAICA. All the operations under warranty (including those of replacement of the product or part thereof) will not prolong the duration of the original period of warranty of the product replaced. The manufacturer declines any liability for any damage that may, directly or indirectly, be caused to persons, property or animals as a result of the non-observance of all the requirements established in the relevant instructions manual and concerning, especially, warnings relating to installation, use and maintenance of the appliance. Laica, in its constant commitment to improving its products, is entitled to changing without any notice, in whole or in part, its products in relation to production requirements, without this entailing any liability for Laica towards its dealers. For further information: www.laica.it.

"UNDERARM USE

For underarm use, proceed as follows: 1) Dry the underarm very well. Before the measurement keep the arm along the hip for 5 minutes to stabilize the temperature. Position the probe under the underarm and keep the arm well pressed against the chest. (This prevents the room temperature, in which one is, from influencing the correct measurement of body temperature).

2) Press the key O/I. The device emits a short acoustic signal. On the display all function symbols do switch on, just for few seconds. Then, the thermometer shows for few seconds the last measurement performed, followed by Lo°C (if the detected temperature is lower than 32°C) or Hi°C (if the detected temperature is greater than 42,9°C). When the symbol °C blinks, the device is ready for use. The temperature stabilizes after about 60 seconds. The standard temperature for this type of measurement is 36,5°C.

STANDARDS

The product corresponds to the below standards: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment — Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential

performance — Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

	Manufactured by: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China
	Shanghai International Corp. GmbH (Europe) Effestrasse 80, 20537 Hamburg, Germany
	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug - Switzerland

Distributed by: Laica S.p.A.
Viale del lavoro, 10 - 36048 Barbarano Mossano (VI)
- Italy - Phone: +39.0444.795314 - www.laica.it
Made in China

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NORMAS

O produto atende às seguintes normas:EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

Prodotto por: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China

Shanghai International Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany

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STANDARDS

Das Produkt erfüllt die folgenden Standards: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

Hergestellt von: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China

Shanghai International Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany

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NORMES

Le produit répond aux normes suivantes: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

Prodot par: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China

Shanghai International Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany

MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug - Switzerland

Distribuído por: **Laica S.p.A.**
Viale del lavoro, 10 - 36048 Barbarano Mossano (VI) – Italy - Phone: +39.0444.795314 - www.laica.it
Made in China

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Immunity aspects			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic disant/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	N/A	
Surge IEC 61000-4-5	±1kV differential mode ±2kV common mode	N/A	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycle 70% UT (30% dip in UT) for 25 cycle <5% UT (>95% dip in UT) for 5 secretary	N/A	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteric of a typical location in a typical commercial or hospital environment.

Guidance and declaration of manufacturer-electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	Test Level EN 60601-1-2	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 Mhz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended seperation distance: $d = \left[\frac{3.5}{E} \right] \sqrt{P}$ 80 MHz to 800 MHz $d = \left[\frac{7}{E} \right] \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

Recommended separation distances between portable and mobile RF communications equipment and the device			
The device is intended for use in an electromagnetic environment in which radiated therefore disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	80 MHz to 800 MHz $d = \left[\frac{3.5}{E} \right] \sqrt{P}$	800 MHz to 2,5 GHz $d = \left[\frac{7}{E} \right] \sqrt{P}$	
0,01	0,12	0,23	
0,1	0,38	0,73	
1	1,2	2,3	
10	3,8	7,3	
100	12	23	

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NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Το προϊόν πληροί τις ακόλουθες προδιαγραφές: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

	Κατοικοσχευέται από: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China
	Shanghai International Corp. GmbH (Europe) Eiffelstrasse 80, 20537 Hamburg, Germany
	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug - Switzerland

Διανέμεται από: **Laica S.p.a. - Viale del lavoro, 10 - 36048 Barbarano Mossano (VI) - Italy Phone: +39.0444.795314 - www.laica.it** Made in China

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STANDARDI

Izdelek ustreza naslednjim standardom: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

	Proizvajalec: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China
EC REP	Shanghai International Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany
CH REP	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug - Switzerland
Distributer: Laica S.p.A. - Viale del lavoro, 10 - 36048 Barbarano Mossano (VI) - Italy - Phone: +39.0444.795314 - www.laica.it	
Made in China	

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STANDARDI

Proizvod ispunjava sljedeće standarde: EN 12470-3 Clinical thermometers —Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device, ISO 80601-2-56 Medical electrical equipment —Part 2-56:Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment —Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60601-1-2(EMC), IEC/EN60601-1(Safety) standards.

	Proizvođač: JOYTECH Healthcare Co., Ltd., No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China
EC REP	Shanghai International Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany
CH REP	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug - Switzerland
Distribuiru: Laica S.p.A. - Viale del lavoro, 10 - 36048 Barbarano Mossano (VI) - Italy - Phone: +39.0444.795314 - www.laica.it	
Made in China	

Guidance and declaration of manufacturer-electromagnetic emissions		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A	


Immunity aspects			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	N/A	
Surge IEC 61000-4-5	±1kV differentail mode ±2kV common mode	N/A	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycle 70% UT (30% dip in UT) for 25 cycle <5% UT (>95% dip in UT) for 5 secretary	N/A	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteritic of a typical location in a typical commercial or hospital environment.

Guidance and declaration of manufacturer-electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	Test Level EN 60601-1-2	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 Mhz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 Ghz	3 V/m	Recommended seperation distance: $d = \left[\frac{3.5}{F} \right] \sqrt{P}$ 80 MHz to 800 MHz $d = \left[\frac{7}{F} \right] \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

Recommended separation distances between portable and mobile RF communications equipment and the device		
The device is intended for use in an electromagnetic environment in which radiated therefore disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.		
Rated maximum output power of transmitter W	80 MHz to 800 MHz $d = \left[\frac{3.5}{F} \right] \sqrt{P}$	800 MHz to 2,5 Ghz $d = \left[\frac{7}{F} \right] \sqrt{P}$
0,01	0,12	0,23
0,1	0,38	0,73
1	1,2	2,3
10	3,8	7,3
100	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.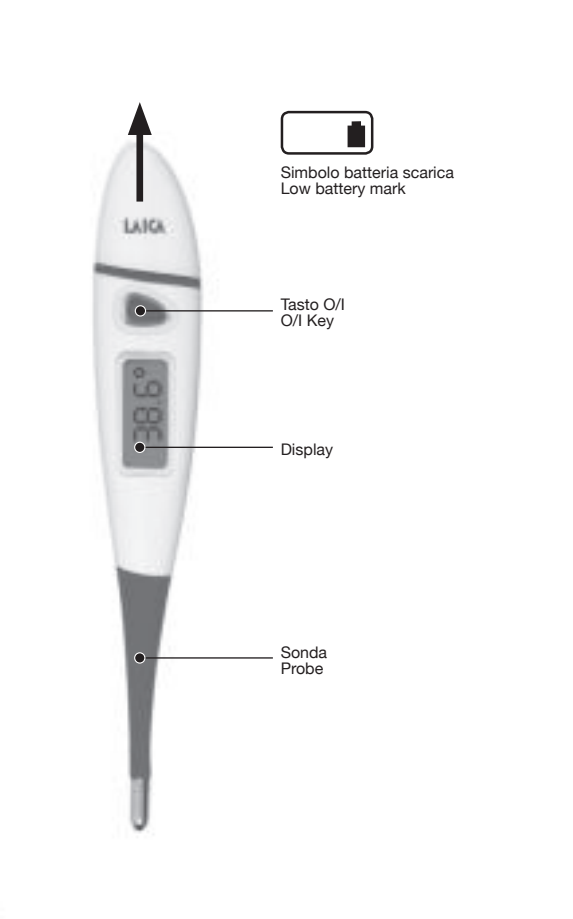

LAICA S.p.A.

Viale del Lavoro, 10
36048 Barbarano Mossano (VI) - Italy
Tel. +39 0444.795314
info@laica.com
Made in China

www.laica.it