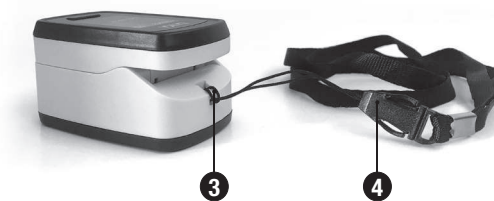
LAICA®

TYPE AS-302-L

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Prodotto da: Shenzhen Acurio Instrument Co., Ltd 502G Part B, Donglian Building, Chuangye 2nd Road, 23rd Subdistrict, Baoan District, Shenzhen, China.

MedNet GmbH - Borkstrasse 10, 48163 Münster, Germany
Phone +49 251 32266-0 Fax +49 251 32266-22

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INSTRUCTIONS AND WARRANTY

IMPORTANT

READ CAREFULLY BEFORE USING - KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

Dear Customer, Laica would like to thank you for choosing this product, designed according to criteria of reliability and quality for complete satisfaction.

IMPORTANT

READ CAREFULLY BEFORE USING - KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

The instruction manual must be considered as part of the product and must be kept for its entire life cycle. If the appliance is transferred to another owner, its documentation must also be transferred in its entirety. To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions. In case of loss of the instruction manual or to receive further information or clarifications contact the company using the details below: LAICA S.p.A. - Viale del Lavoro, 10 - 36048 Barbarano Mossano (VI) - Italy - Phone +39 0444.795314 - info@laica.com - www.laica.it

Intended use and scope of application: The finger pulse oximeter is a non-invasive device that allows measurement of arterial oxygen saturation (SpO2) and heart rate (pulse rate). It can be applied to all persons except children and infants. Oxygen saturation indicates the percentage of haemoglobin linked to the oxygen in arterial blood. Saturation is measured by a method called pulse oximetry; the device uses two beams of light with different wavelengths that strike the finger that is inserted into the device.

KEY TO SYMBOLS	
	Warning
	Attention! Read the user instructions carefully
	In compliance with European legislation on medical devices
	Manufacturer
	Production batch number
	European representative
	Serial number

IP21: Degree of protection of enclosures for electrical equipment, where the first digit indicates the degree of protection against the penetration of solid foreign bodies (from 0 to 6) and the second digit the degree of protection against the penetration of liquids (from 0 to 8).

SAFETY WARNINGS

- Prior to use, ensure the product is intact and that there is no visible damage. If in doubt, contact the retailer.
- Keep the plastic bag away from children: danger of suffocation.
- This product must be used only for its intended purpose and according to the user instructions. All other types of use shall be considered inappropriate and therefore hazardous. The manufacturer shall not be deemed liable for any damage caused by inappropriate or incorrect use.
- The use and maintenance of this product is only allowed by persons aged 14 years or over (capable of reading and understanding the instructions for use), and only

- if properly supervised by an adult in the case of persons with reduced physical, sensory or mental capacities, or unskilled persons. Children must not play with the device.
- To avoid any possibility of accidental strangulation, keep the strap away from children.
- Treat the product with care, protect it from impacts, extreme variations in temperature, humidity, dust, direct sunlight and sources of heat.
- In the event of a failure and/or malfunction, turn off the device without tampering with it. Always contact the retailer for repairs.
- Make sure your hands are dry when using the device. **NEVER submerge the product in water or any other liquids.**
- Device for domestic use that is not to be used outdoors.

ATTENTION! BEFORE USING THIS DEVICE

- Self-measurement means checking, not diagnosis or treatment. Unusual readings must always be discussed with your physician. Under no circumstances should you alter the dosages of any medication prescribed by your physician.**
 - DO NOT look directly at the sensors for measurement (red light and invisible infrared light): these lights are harmful to the eye.
 - The device has no alarm functions.
 - Device is NOT suitable for continuous monitoring.
 - DO NOT use the pulse oximeter for more than 2 hours on the same finger.
 - Prolonged use of the device may be painful in people suffering from circulation disorders.
 - The device may provide inaccurate readings when used in temperature or humidity conditions outside the limits indicated in the "Technical Specifications" paragraph.
 - Do not use near strong magnetic fields and therefore keep away from radio or mobile phone systems (for more information on the interferences, see the paragraph "Electromagnetic compatibility").
 - DO NOT use the device in the presence of a flammable anaesthetic mixture with air, oxygen or nitrous oxide.
 - DO NOT use this device if you are allergic to plastic and/or rubber.
 - If the accuracy of the measurement by the device is uncertain, contact your doctor.
 - Errors or a reduction in the accuracy of measurement may occur in the following cases:
 - incorrect finger placement in the pulse oximeter;
 - nails that are too long that do not allow the finger to be inserted fully into the device;
 - nails with polish or the presence of fake nails;
 - hand movements during measurement;
 - measurement in subjects suffering from heart rate disorders;
 - in the event of carbon monoxide poisoning;
 - use of the device in subjects with low blood pressure, who suffer from jaundice or who take constrictive drugs or in subjects who are regular smokers.
- #### SAFE USE OF BATTERIES
- Remove the batteries if the device is not used for long periods of time and keep them in a well-ventilated and dry place, at room temperature.
 - DO NOT use rechargeable batteries.
 - NEVER expose the batteries to sources of heat or direct sunlight. Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
 - DO NOT throw the batteries into an open flame.
 - Only adults should remove or replace the batteries.
 - Keep batteries out of reach of children: the ingestion of batteries can cause serious harm and death. If a battery is swallowed, seek urgent medical care.
 - The acid in batteries is corrosive. Avoid contact with the skin, eyes or garments.

PRODUCT DESCRIPTION (see fig. 1)

- Power button
- Finger clip
- Strap socket
- Strap
- LED display
 - Oxygen saturation in percent
 - Heart beats per minute
 - Heart beat column
- Batteries

TECHNICAL SPECIFICATIONS

- Product name: Finger pulse oximeter
- Trade name: EA1007
- Classification: Internal energy, class BF applied part
- LED display
- SpO2:
 - Measurement range: 36%-99%
 - Resolution: 1%
 - Precision: $\pm 2\%$ (70% ~ 99%), <70% not definable
- Pulsations:
 - Measurement range: 30bpm ~ 250bpm
 - Resolution: 1bpm
 - Accuracy: the greater between $\pm 1\%$ and 1 beat
- Power supply: 2 AAA 1.5V alkaline batteries
- Operating current: 20mA ~ 130mA
- Sensors for measurement: Red light (wavelength 662nm ~ 666 nm 7mW), infrared light (wavelength 890nm ~ 900nm 5.5mW)
- Data update cycle: not more than 12 seconds

- Life expectancy of the product in use (used 15 times a day for approximately 10 minutes): 5 years.
- Operating conditions:** temperature: min. 5°C - max. 40°C, with air humidity $\leq 80\%$ and atmospheric pressure of 86 to 106 kPa;
- Storage conditions:** temperature: min. -20°C - max. 55°C, with air humidity $\leq 93\%$ (without condensation) and atmospheric pressure of 50 to 106 kPa.

INSTRUCTIONS FOR USE

INSERTION/REPLACEMENT OF THE BATTERY

This device operates with 2 AAA 1.5V alkaline batteries. At the first use, and when the display shows the battery symbol, proceed with the insertion and/or replacement of the batteries. Open the battery compartment by sliding the cover in the direction of the arrow, then insert the batteries according to the indicated poles and close the cover. Dispose of used batteries as indicated in the "Disposal procedure" paragraph.

USE

- Before using the device for the first time, clean it as described in the "Maintenance and disinfection" section.
- Before each use, disinfect the finger and rubber portion of the pulse oximeter (inside the finger clip) using medical alcohol.
- Insert your finger into the clip of the device. Keep the finger still.
- Press the power button: the pulse oximeter starts measuring. Do not move during readings. After a few seconds, the display shows the measurement values in real time.
- When the measurement is complete, remove your finger: the pulse oximeter switches off automatically after approximately 20 seconds.

MAINTENANCE AND DISINFECTION

Before and/or after each use, clean the rubber part inside the finger clip with alcohol, taking care not to damage the measurement sensors. Allow the device to dry completely before using it. Clean the outside of the pulse oximeter using a soft, non-abrasive cloth. Never use abrasives and never submerge the device in water or other liquids. Keep the pulse oximeter in a clean, dry place, away from direct sunlight. Remove the batteries when the device is not being used for long periods of time.

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not turn on.	The batteries are worn.	Replace the batteries.
	The batteries have not been installed correctly.	Check the polarity of the batteries during insertion.
The display shows	The batteries are flat.	Replace the batteries.
The measurements differ greatly from those previously taken.	Your finger is not inserted correctly.	Position your finger firmly in the pulse oximeter.
The device displays erratic data.	The vascularisation of the finger is insufficient for measurement.	Reread the warnings and use a different finger for the measurement.
	The finger used for the measurement is too large or too small.	Change the finger being used for the measurement.
	A movement was made during the reading.	Keep still during the reading.

N.B. If the device does not resume its correct functioning despite the checks performed, contact your retailer.

DISPOSAL PROCEDURE (Dir. 2012/19/EU-WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

To correctly dispose of batteries (Dir. 2013/56/EU) do not throw them into household waste but rather dispose of them as special waste at specialised recycling centres. For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means of the **stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.** This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects

and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to wear and tear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA. All interventions covered by warranty (including g product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

Manufactured by: Shenzhen Acurio Instrument Co., Ltd 502G Part B, Donglian Building, Chuangye 2nd Road, 23rd Subdistrict, Baoan District, Shenzhen, China.

MedNet GmbH - Borkstrasse 10, 48163 Münster, Germany
Phone +49 251 32266-0 Fax +49 251 32266-22

Distributed by: Laica S.p.A.
Viale del lavoro, 10 - 36048 Barbarano Mossano (VI) – Italy
Phone: +39.0444.795314 - info@laica.com - www.laica.it
Made in China

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